

Indication for Use

1. Post surgical and injectable rhinoplasty
2. After nose injury or trauma including fractured nasal bone.
3. To reshape and enhance appearance of the nose.



Figure 1



Figure 2



Figure 3



Figure 4



Figure 5



Figure 6



Figure 7

Instructions for Use

Single Patient Use

Warning Do not use directly on broken skin.

Before removing the backing from the adhesive, apply the splint to the nose to determine the ideal size and position for the splint. **Figure 1.**

Remove the backing from the adhesive. **Figure 2.**

Clean the skin, and when dry, apply the splint to the nose. **Figure 3.**

The splint can be removed and reapplied to adjust its position if required.

To reduce the grip, position the splint a little higher on the nose (toward the eyes). **Figure 4.**

To increase the grip, position the splint a little lower on the nose (toward the nostrils). **Figure 5.**

The nostrils can be protected or not depending on the indication.

The splint can be forced open a few times to reduce the grip, if it is too tight. **Figure 6.**

A self-adhesive Dorsal pad can be used under the splint to compress the Nasal Dorsum if needed. **Figure 7.**

www.lemed.com.au

EC Representative: Priory Analysts Ltd, The Pinnacle, 160 Midsummer Boulevard Milton Keynes MK9 1FF, UK

Manufactured and Sponsored in Australia by:

Logikal Health Products Pty Ltd, PO Box 218 Morisset NSW 2264

CE 0805

2-4000 Rev.0